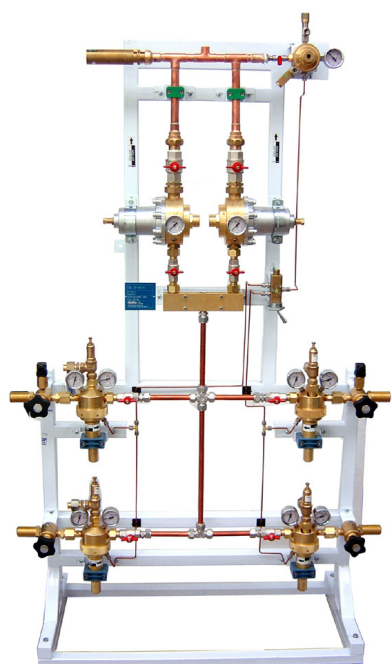
Automatic central unit with
4 regulators 2210 and 2 regulators 2250

Centralina automatica
con 4 rid. 2210 e 2 rid. 2250

C-2C2210M-2250M

Automatic central unit
with 4 regulators 2210 and 2 regulators 2250

Centralina automatica
con 4 rid. 2210 e 2 rid. 2250

C-2C2210M-2250M

Automatic changeover supply unit for medical gases high flow rate. It is composed of four high pressure valves with metal seat and metal plug, four pressure reducers **2210M series**, two pressure reducers **2250M series** and a pneumatic system for the changeover between the ramp in dispensing and the secondary source and vice versa.

Two pressure reducers are connected to the output of the same line. When they are adjusted to the same pressure, they work simultaneously. Otherwise the pressure regulator set to the higher pressure, inhibits the operation of the other one.

The decompression supply unit works by exploiting this characteristic, in fact pressure-reducers **2210M series** are adjusted with a mixed system both a Spring and pneumatic system, which allows to keep in dispensing the pressure regulator set to the higher pressure.

When the inlet pressure to the pair of pressure reducers **2210M series** in operation, falls below the minimum pressure value exchange, automatically comes into operation the other pair of pressure reducers **2210M series**. The operator, during the replacement of empty cylinders will reactivate the automatic system increasing, through the appropriate command, the output pressure of the pressure reducer in delivery. The supply unit was provided of a by-pass system, with the aim of ensuring the gas supply even in the case become necessary maintenance interventions, in fact each pressure reducer and the driving part can be isolated (thanks to the special ball valves) allowing the partial operation of the supply unit and thus ensure the continuity of supply. The supply unit is suitable for use with medical gases.

The main features of this model are:

- The use of two first-stage reducers mounted in parallel, which allows to guarantee high flow delivery (up to 200 Nm³ / h)
- First stage and second stage Pressure Reducers in accordance with **ISO 10524-2** standard
- Shut-off valves with high pressure metal plug and seat, these valves have passed the adiabatic compression test as required by **ISO 7396-1**.
- High and low pressure gauges according to **EN 837-1**
- Pressure of high pressure pre-calibrated at 20 bar, have passed the test of adiabatic compression
- System by-pass
- Incoming Connections varied according to gas

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Automatic central unit with 4 regulators 2210 and 2 regulators 2250 **C-2X2210M-2250M**

Technical Data

SUITABLE FOR THE FOLLOWING GASES: O₂, N₂O, Air, N₂, CO₂

MAX FLOW RATE: 200 Nm³/h

MAX INLET PRESSURE: 200 bar (for O₂, Air and N₂); 110 bar (for N₂O and CO₂)

MIN INLET PRESSURE: 20 bar

OUTLET PRESSURE: 8 bar

STORAGE TEMPERATURE: -20 ÷ +60 °C

OPERATING TEMPERATURE: 0 ÷ 50 °C

OUTLET CONNECTION: G1/2 M

HIGH PRESSURE SWITCH CONNECTIONS: G1/8 F

PRESSURE RELIEF VALVE OUTLET CONNECTION: soldering coupler ø 16mm

CONTROL REGULATOR PRESSURE RELIEF VALVE OUTLET: soldering coupler ø 8mm

DIMENSIONS (mm): 1110x1750

FRAME MATERIAL: Tubular 40mm x 40mm black painted


STANDARDS: ISO 10524-2; ISO 21969; ISO 15001; ISO 7396-1 ; EN 837-1

ALL THE COMPONENTS OF THE SYSTEM SUBJECTED TO CYLINDER PRESSURE OR TO SINGLE FAULT CONDITION HAVE SUCCESSFULLY WITHSTOOD THE ADIABATIC COMPRESSION TEST IN ACCORDANCE WITH ISO 10524-2 AND ISO 15001


DESGREASED IN ACCORDANCE WITH ISO 15001

Gas Table	GAS	GAS SYMBOL	COLOR	HIGH PRESSURE CONNECTION
	Oxygen	O ₂	White	UNI 11144 (W 21,7 M x 1/14")
	Medical Air	AIR	White / Black	UNI 11144 (W 30 M x 1/14")
	Nitrous Oxide	N ₂ O	Blue	UNI 11144 (G 3/8" M)
	Carbon Dioxide	CO ₂	Grey	ISO 5145 (W 27 x 2 M)
	Nitrogen	N ₂	Black	UNI 11144 (W 21,7 F x 1/14")

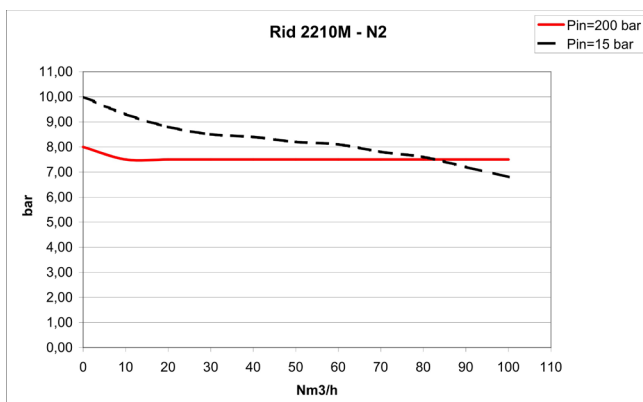Pressure regulator 2210 M flow rate diagram

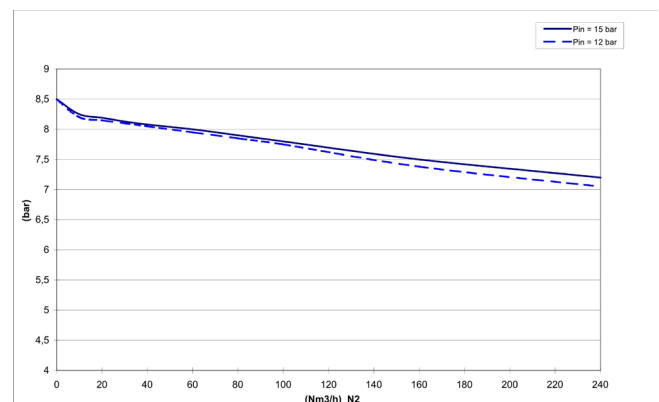Pressure regulator 2250 M flow rate diagram

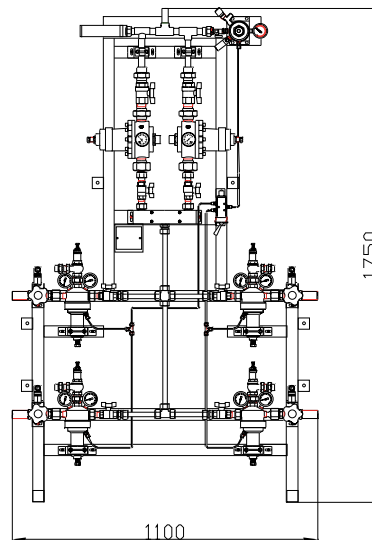
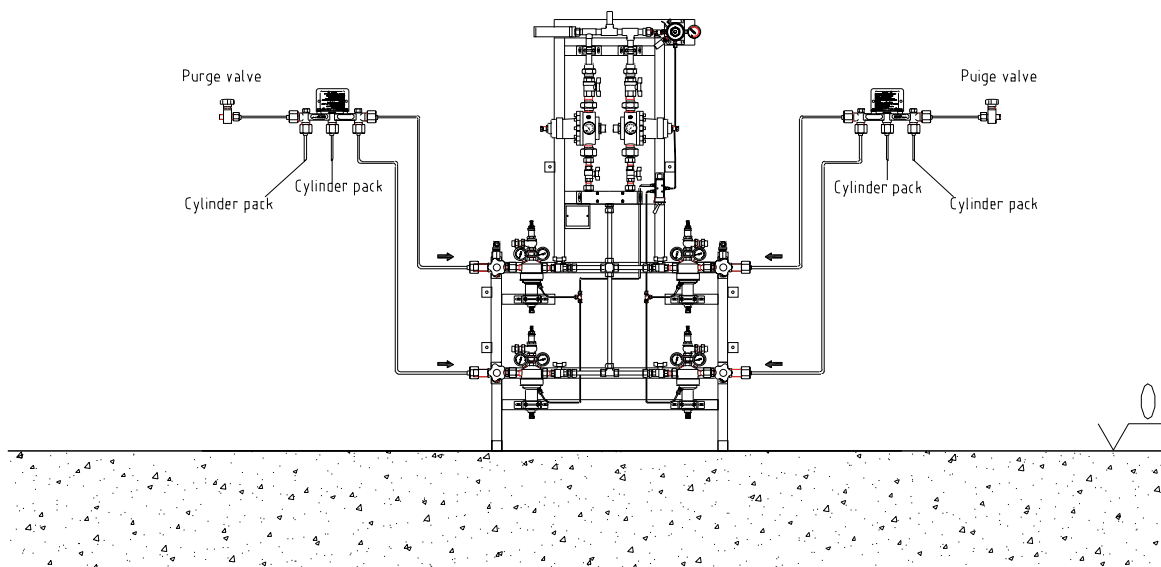
Automatic central unit with 4 regulators 2210 and 2 regulators 2250 **C-2X2210M-2250M**

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